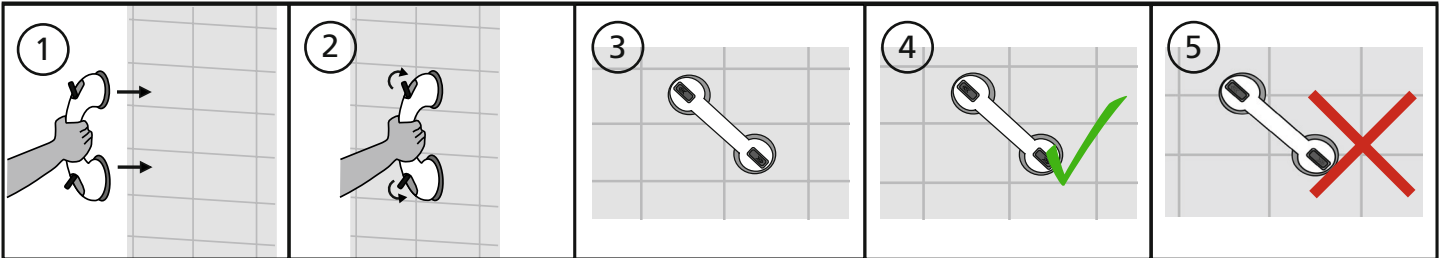
- Ⓒ Gebrauchsanweisung
- ⒸⒹ Instructions for use
- Ⓕ Mode d'emploi
- Ⓒ Instrucciones para el uso
- Ⓒ Istruzioni per l'uso
- ⒸⒹ Gebruiksaanwijzing

- Ⓕ Instruções de utilização
- ⒸⒹ Brugsanvisning
- Ⓕ Bruksanvisning
- ⒸⒹ Instrukcja obsługi
- ⒸⒹ Οδηγίες χρήσης
- ⒸⒹ Návod k použití

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Ⓒ **Bathroom wall grab handle TRAFFIC LIGHT SYSTEM**

The red/green traffic light system on each suction pad indicates whether the handle has a firm hold (green) or whether the suction pad needs to be re-positioned (red).

Instructions for use

Place the bathtub safety handle in the desired position on the bath or shower wall. Press the handle with the suction cups against the sur-face. Make sure that the suction cups are not positioned on a joint. Push the two snap-in fittings toward the wall to additionally increa-se the vacuum and close the snap-in fittings. Dismantling takes place in reverse order.

IMPORTANT!

1. SUITABLE FOR MOUNTING ON ALL AIR-IMPERMEABLE SURFACES ONLY (SUCH AS E.G. TILES, GLASS, PLASTIC-COATED WOODEN PANELS).
2. Make sure that the area on which you wish to fix your suction pad is clean dry and absolutely grease-free. Clean the surface and allow it to dry before mounting. The bottom (suction surface) of the suction pad must also be absolutely clean.

ATTENTION! The handle is only intended to help keep your balance. It must under no circumstances be use to support the total body weight. Check the seating of the handle before each use and re-fix if necessary.

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